

RelyX™ Ultimate

Adhesive Resin Cement



STEP-BY-STEP

3M ESPE

3M ESPE

RelyX™ Ultimate

Adhesive Resin Cement

8,5g Base Paste

440075814



INDICATIONS

ETCHABLE GLASS CERAMICS – OXIDE CERAMICS – COMPOSITE – METAL



Inlay



Onlay



Crown



Bridge



Maryland Bridge and Inlay/Onlay Bridge



Veneer



Endodontic Post



Restoration to implant abutment

INDICATIONS

INTEGRATED COMPONENTS



Scotchbond™ Universal Adhesive*


*Scotchbond™ Universal Adhesive functions not only as a self-etch or total-etch adhesive, but also as a primer for metal and zirconia restorations as well as a silane for glass ceramics.

PRE-TREATMENT OF ETCHABLE GLASS CERAMICS

1



2

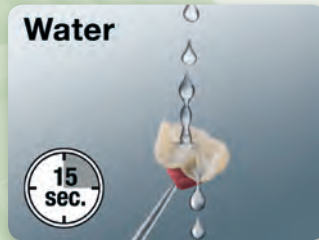

3



4




5



PRE-TREATMENT

PRE-TREATMENT OF OXIDE CERAMICS & METAL & COMPOSITES

1

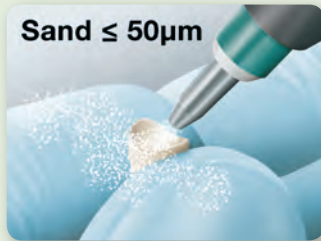


2



3



4

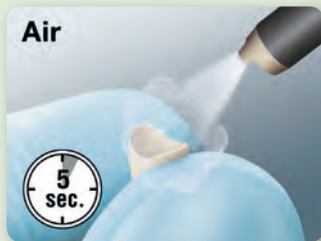

PRE-TREATMENT

PRE-TREATMENT OF FIBER POST

1



2



3



4



PRE-TREATMENT

SELF-ETCH TECHNIQUE

1



2



3



4



5



6



7



8



CEMENTATION

PE
elyX™ Ultimate
ve Resin
ment

9



10



11



12



PROCEDURE

TOTAL-ETCH OR SELECTIVE-ENAMEL-ETCH TECHNIQUE


1



2



3

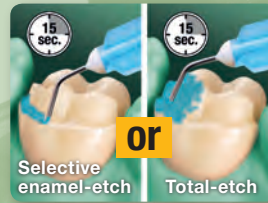

4



5



6



7



8



CEMENTATION

9



10



11



12



13




or



14



or

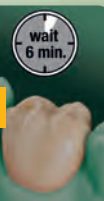

15



PROCEDURE

MARYLAND AND INLAY / ONLAY BRIDGE

1



2



3



4



5



6



7



8



CEMENTATION

9



10




11



12




13

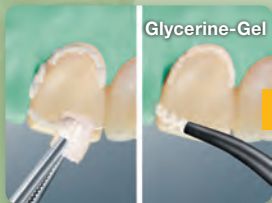

or



14

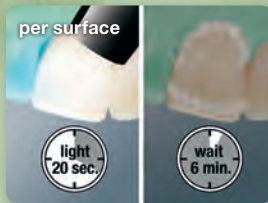


15




PROCEDURE

FIBER POST


1

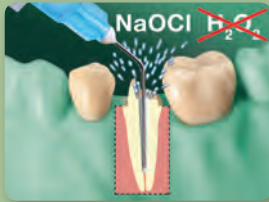

2

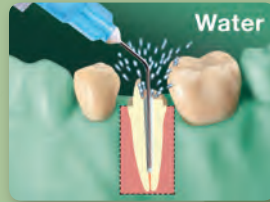

3

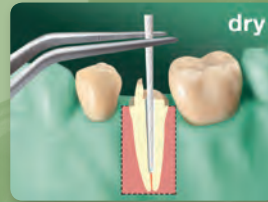

4




5



6



7

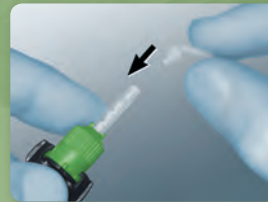


8




CEMENTATION

PE
elyX™ Ultimate
ve Resin

9

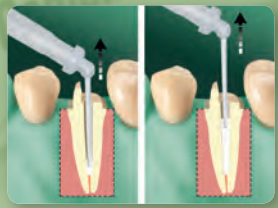

10

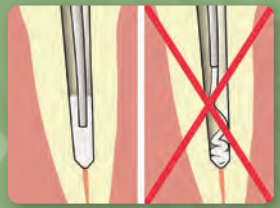

11




12



13




14

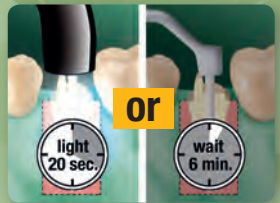

PROCEDURE

RelyX™ Ultimate

 Adhesive Resin Cement

www.3MESPE.com

3M ESPE

Dental Products

3M Center
Building 275-2SE-03
St. Paul, MN 55144-1000
U.S.A.
1-800-634-2249

3M Canada

Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-888-363-3685

3M, ESPE, RelyX and Scotchbond are trademarks of 3M or 3M Deutschland GmbH. Used under license in Canada.
© 3M 2012. All rights reserved.

Please recycle. Printed in U.S.A.
70-2013-0510-2